



Measuring Guidelines for TributeNight Facial Garments

Positioning Your Client When Measuring for Facial Garments

Tribute garments are most commonly worn while the client is sleeping, however, clients with facial swelling may be measured in a seated posture.

Measuring:

Please record the needed measurements in the designated areas on the Facial Order Form.

- A:** Measure the length from one ear tip to the other.
- B:** Measure the head circumference at the temple line or largest part.
- C:** Measure the distance from the outside crease of the right eye to the outside crease of the left eye.
- D:** Measure the length of the nose.
- E:** Measure the length at the midline from the nose to the upper lip.
- F:** Measure the length at the midline from the lower lip to the chin.
- G:** Measure the length from the sterno-clavicular notch to the chin. Follow the contour with chin in relaxed posture.
- H:** Measure from the right ear lobe to the right nostril.
- I:** Measure from the left ear lobe to the left nostril.
- J:** Measure the distance from the right TMJ to the chin.
- K:** Measure the distance from the left TMJ to the chin.
- L:** Measure the length from the right ear lobe to the HPS (High Point of Shoulder).
- M:** Measure the length from the left ear lobe to the HPS.
- N:** Measure the neck circumference.

Fax to **443-455-1402**