

# sigvaris

# Legassist®

**CUSTOM CALF & FOOT**

## CUSTOM Measuring Instructions

### Important

Exact measurements are critical for this garment to ensure a proper fit. If you have questions about measuring for this garment or would like assistance with measuring and fitting this garment please contact

### Supplies needed

- Cell phone with camera. Photos of the arm with measurement markings must be emailed to:
- SIGVARIS GROUP Measuring tape and body pen (or eyeliner pencil).
- Measuring instructions and forms.

## INSTRUCTIONS

All measurements should be recorded in centimeters. Apply slight tension to hold the tape measure in place.

### STEP 1: Locate Landmarks

1. The ideal placement for the top edge of the garment is two finger widths below the popliteal crease, and across the fibular head and tibial tuberosity. However, the position of the top of garment can be customized, if needed, to overlap better with a Legassist LCS thigh piece. Encircle leg with top edge of tape measure at the desired location. Using the SIGVARIS GROUP body pen, trace a line around the leg along the top edge of the tape and measure and record the circumference on top of garment line.
2. Encircle leg with bottom edge of tape measure aligned with the ankle bend. Mark leg on this edge of tape at the anterior, medial, posterior and lateral aspects, and record the circumference on the line labeled ankle bend (ø point).

### STEP 2: Locate Circumference Measurement Positions

1. Starting with "0" of the tape measure at your mark on the lateral aspect of the ankle bend, mark the leg in 5cm increments up to your mark on the lateral aspect of the top of garment.

### STEP 3: Measure Lengths

1. Measure leg from your mark on the anterior aspect of the ankle bend the anterior aspect of top of garment and record length on line marked A.
2. Measure leg from your mark on the medial aspect of the ankle bend to the medial aspect of top of garment and record length on line marked B.
3. Measure leg from your mark on the posterior aspect of the ankle bend to the posterior aspect of top of garment and record length on line marked C.
4. Measure leg from your mark on the lateral aspect of the ankle bend to the lateral aspect of top of garment and record length on line marked D.

### STEP 4: Measure Circumferences

1. Encircle leg with a tape measure at each mark located between the ankle bend and top of garment, and record each circumference on the line corresponding to each position.

### STEP 5: Measure Foot

1. Measure foot from heel to 1st metatarsal head (or desired boot length) and record length on line K.
2. Measure top of foot from the 3rd metatarsal head to ankle bend and record length on line M.
3. Encircle the instep of foot and apex of heel with a tape measure and record the circumference on line I.
4. Encircle the foot across the metatarsal heads and record the circumference on Line J.

NOTE: Legassist Calf & Foot is available in:  
7099-BK Flat Foam, 7199-BK WaveFoam

# sigvaris

# Legassist®

## CUSTOM CALF & FOOT

|                      |               |               |            |         |
|----------------------|---------------|---------------|------------|---------|
| Patient Name:        | Contact Name: | Height:       | Weight:    | Date:   |
| Ship Name & Address: |               | Account Name: | Account #: | P.O #.: |

### Product Information

Product includes one Legassist Calf and One Compreboot Plus Foot.

|                                                                                                             |                                                                                                            |                                                                            |                                                                                        |
|-------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------|----------------------------------------------------------------------------------------|
| <input type="checkbox"/> Right Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Left Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Foam<br><input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | Additional Foot Option<br><input type="checkbox"/> Custom Medaboot (additional charge) |
| <input type="checkbox"/> Right Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Left Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Foam<br><input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | Additional Foot Option<br><input type="checkbox"/> Custom Medaboot (additional charge) |

### Note:

If the greatest circumference measurement is >60cm, order a Super.

### Circumference Measurements

(Lateral aspect of leg)

**Top of Garment** \_\_\_\_\_

40cm \_\_\_\_\_

35cm \_\_\_\_\_

30cm \_\_\_\_\_

25cm \_\_\_\_\_

20cm \_\_\_\_\_


15cm \_\_\_\_\_

10cm \_\_\_\_\_

5cm \_\_\_\_\_

**Ø (Point Ankle Bend)** \_\_\_\_\_

\* Choose a zero point distance from floor

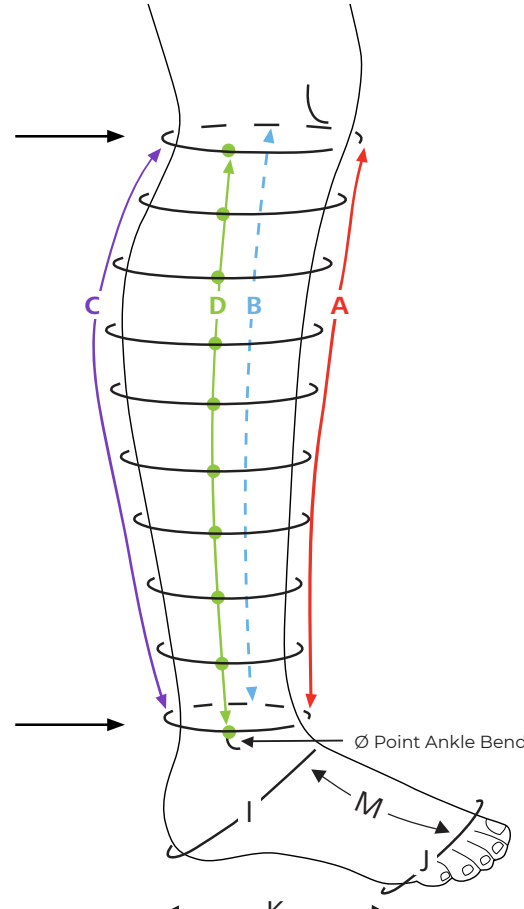

### Length Measurements

**Anterior**

A \_\_\_\_\_

**Medial**

B \_\_\_\_\_

**Posterior**

C \_\_\_\_\_

**Lateral**

D \_\_\_\_\_

### Foot Measurements

I \_\_\_\_\_

J \_\_\_\_\_

K \_\_\_\_\_

M \_\_\_\_\_

# sigvaris

# Legassist®

## CUSTOM CALF & FOOT WITH SHELF

|                      |               |               |            |         |
|----------------------|---------------|---------------|------------|---------|
| Patient Name:        | Contact Name: | Height:       | Weight:    | Date:   |
| Ship Name & Address: |               | Account Name: | Account #: | P.O #.: |


### Product Information

Product includes one Legassist Calf and One Compreboot Plus Foot.

|                                    |                                                                 |                                                              |
|------------------------------------|-----------------------------------------------------------------|--------------------------------------------------------------|
| <input type="checkbox"/> Right Leg | Foam                                                            | Additional Foot Option                                       |
| <input type="checkbox"/> Left Leg  | <input type="checkbox"/> Flat <input type="checkbox"/> Wavefoam | <input type="checkbox"/> Custom Medaboot (additional charge) |

### Note:

If the greatest circumference measurement is >60cm, order a Super.

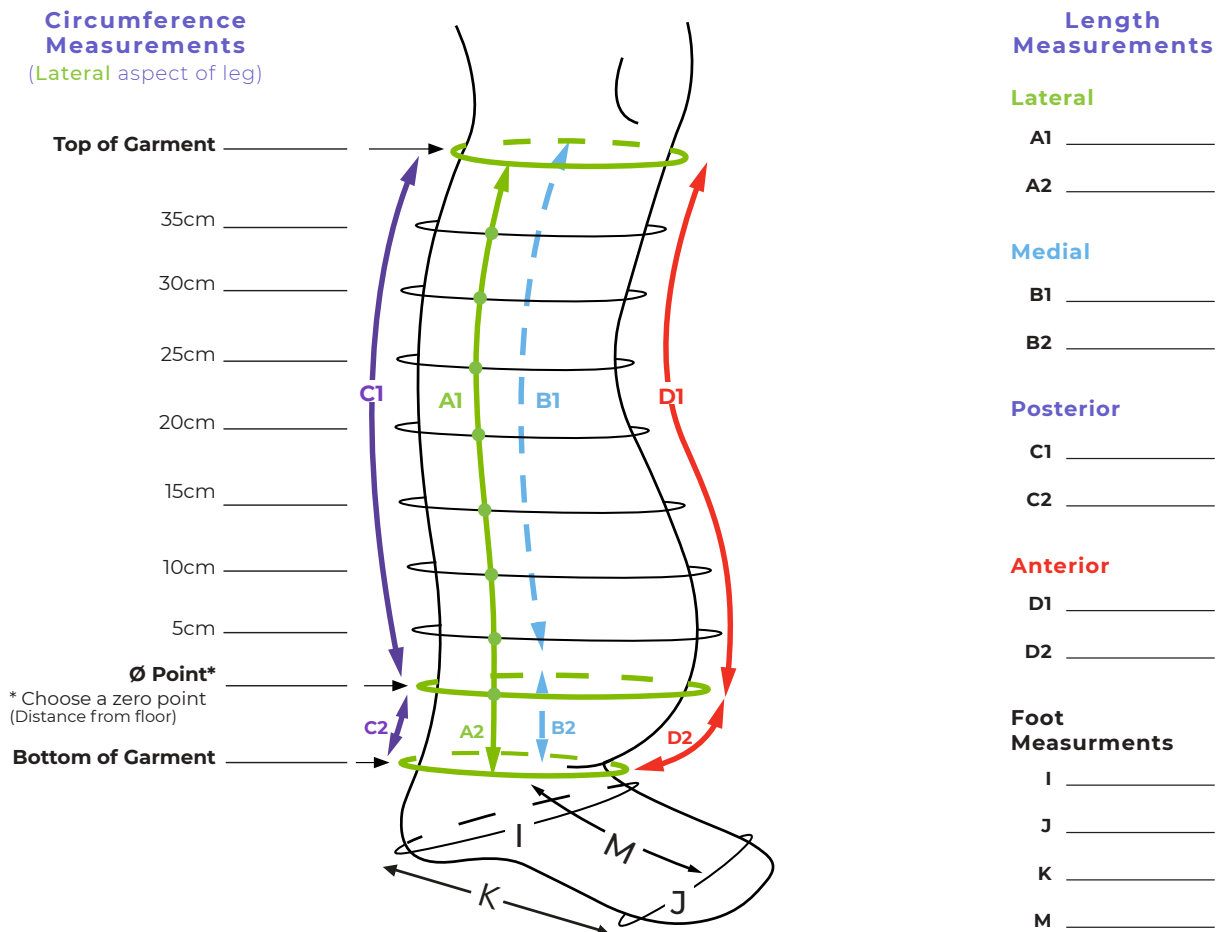

sigvaris

Legassist®  
CUSTOM CALF & FOOT WITH SHELF

## CUSTOM Measuring Instructions

### Important

Exact measurements are critical for this garment to ensure a proper fit. If you have questions about measuring for this garment or would like assistance with measuring and fitting this garment please contact

### Supplies needed

- Cell phone with camera. Photos of the arm with measurement markings must be emailed to:
- SIGVARIS GROUP Measuring tape and body pen (or eyeliner pencil).
- Measuring instructions and forms.

## INSTRUCTIONS

All measurements should be recorded in centimeters. Apply slight tension to hold the tape measure in place.

### STEP 1: Choose The Zero Point (Ø point)

The zero point is unique to the Legassist Calf & Foot with Shelf garment. It is the reference point from which all other measurements are taken, in order to determine the lengths needed for the custom garment to fit the patient's individual ankle shape. Encircle leg with bottom edge of tape measure immediately above the shelf, at the lowest aspect of the curve of the ankle lobule where a tape measure can be applied without sliding off. Trace a line around the leg along the bottom edge of the tape using the SIGVARIS GROUP body pen, and record the circumference on Ø point line. Document your zero point location by measuring the distance from the floor.

### STEP 2: Choose Where The Garment Will End Distally - "bottom of garment"

Encircle leg with bottom edge of tape measure where the ankle lobule touches the ankle and/or foot. Do not pull tape under shelf. Mark leg on this edge of tape at the anterior, medial, posterior and lateral aspects, and record the circumference on the line labeled bottom of garment. The smallest ankle measurement is not relevant for this product. The product is designed to hug and lift the ankle lobule.

### STEP 3: Choose Where The Garment Will End Proximally - "top of garment"

The ideal placement for the top edge of the garment is two finger widths below the popliteal crease, and across the fibular head and tibial tuberosity. However, the position of the top of garment can be customized, if needed, to overlap better with a Legassist LCS thigh piece. Encircle leg with top edge of tape measure at the desired location. Using the SIGVARIS GROUP body pen, trace a line around the leg along the top edge of the tape and measure and record the circumference on top of garment line.

### STEP 4: Measure And Record Lengths

Measure each of the lengths below from the Zero Point, following the contour of the leg and ankle, but not extending the tape measure into folds. Lengths measured distal to the Zero Point should end where the ankle lobule touches the ankle or foot at the bottom of garment versus extending the tape measure up under the lobule. Record lengths as follows:

#### Lateral (A) Lengths:

A1: Measure & record lateral length from Ø point to top of garment

A2: Measure & record lateral length from Ø point to bottom of garment

#### Medial (B) Lengths

B1: Measure & record medial length from Ø point to top of garment

B2: Measure & record medial length from Ø point to bottom of garment

#### Posterior (C) Lengths

C1: Measure & record posterior length from Ø point to top of garment

C2: Measure & record posterior length from Ø point to bottom of garment

#### Anterior (D) Lengths

D1: Measure & record anterior length from Ø point to top of garment

D2: Measure & record anterior length from Ø point to bottom of garment

# sigvaris

# Legassist®

**CUSTOM CALF & FOOT WITH SHELF**

## CUSTOM Measuring Instructions

### STEP 5: Measure And Record Circumferences

---

On the lateral aspect of the leg only, mark every 5cm moving proximally from the zero point. Next, record the circumferences at each point, making sure that the measuring tape is parallel to the floor.

### STEP 6: Measure Foot

---

1. Measure the circumference from where the shelf touches instep of the foot to the apex of the heel and record on line I.
2. Measure the foot circumference at the metatarsal heads and record on line J.
3. Measure the length from the 1st metatarsal head to the posterior aspect of the heel, and record on line K.
4. Measure the length from the dorsal aspect of the third metatarsal head to where the shelf touches the instep of the foot and record on line M.

### STEP 7: Pictures

---

Now that you have completed your measurements, take pictures of the leg (front, back, side) that show all of the measurement markings, as well as the overall shape of the leg. This will assist our expert design team to further customize the garment according to your patient's individual needs. Email the pictures with your completed forms to:

the garment.

### STEP 8: Select Additional Options

---

You are ready to complete your forms and place the order.

Choose right or left leg

Choose Size: If the greatest leg circumference is >60cm, order a Super.

Choose Flat or Wavefoam (recommended)

Choose Compreboot Plus (included) or Custom Medaboot (additional charge)

NOTE: Legassist Calf & Foot With Shelf is available in:

7099-BK Flat Foam, 7199-BK WaveFoam, 7299-BK Super, Flat Foam, 7399-BK Super, WaveFoam.

sigvaris

Legassist®  
FULL LEG

|                      |               |            |         |       |
|----------------------|---------------|------------|---------|-------|
| Patient Name:        | Contact Name: | Height:    | Weight: | Date: |
| Ship Name & Address: | Account Name: | Account #: | P.O #.: |       |

### Product Information

Product includes one Legassist Full Leg and one Compreboot Plus Foot.

|                                                                                                             |                                                                                                            |                                                                            |                                                                                        |                                                     |
|-------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------|----------------------------------------------------------------------------------------|-----------------------------------------------------|
| <input type="checkbox"/> Right Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Left Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Foam<br><input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | Additional Foot Option<br><input type="checkbox"/> Custom Medaboot (additional charge) | Optional<br><input type="checkbox"/> Hip Attachment |
| <input type="checkbox"/> Right Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Left Leg<br>Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Foam<br><input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | Additional Foot Option<br><input type="checkbox"/> Custom Medaboot (additional charge) | Optional<br><input type="checkbox"/> Hip Attachment |

### Note:

If the greatest circumference measurement is >90cm, order a Super.

### Circumference Measurements

● = Locations measured along Lateral aspect of leg

Top of Garment \_\_\_\_\_

35cm \_\_\_\_\_  
30cm \_\_\_\_\_  
25cm \_\_\_\_\_  
20cm \_\_\_\_\_  
15cm \_\_\_\_\_  
10cm \_\_\_\_\_  
5cm \_\_\_\_\_


Ø Point \_\_\_\_\_

Mid Patella \_\_\_\_\_

Ø Point \_\_\_\_\_

5cm \_\_\_\_\_  
10cm \_\_\_\_\_  
15cm \_\_\_\_\_  
20cm \_\_\_\_\_  
25cm \_\_\_\_\_  
30cm \_\_\_\_\_  
35cm \_\_\_\_\_  
40cm \_\_\_\_\_

Ankle Bend \_\_\_\_\_

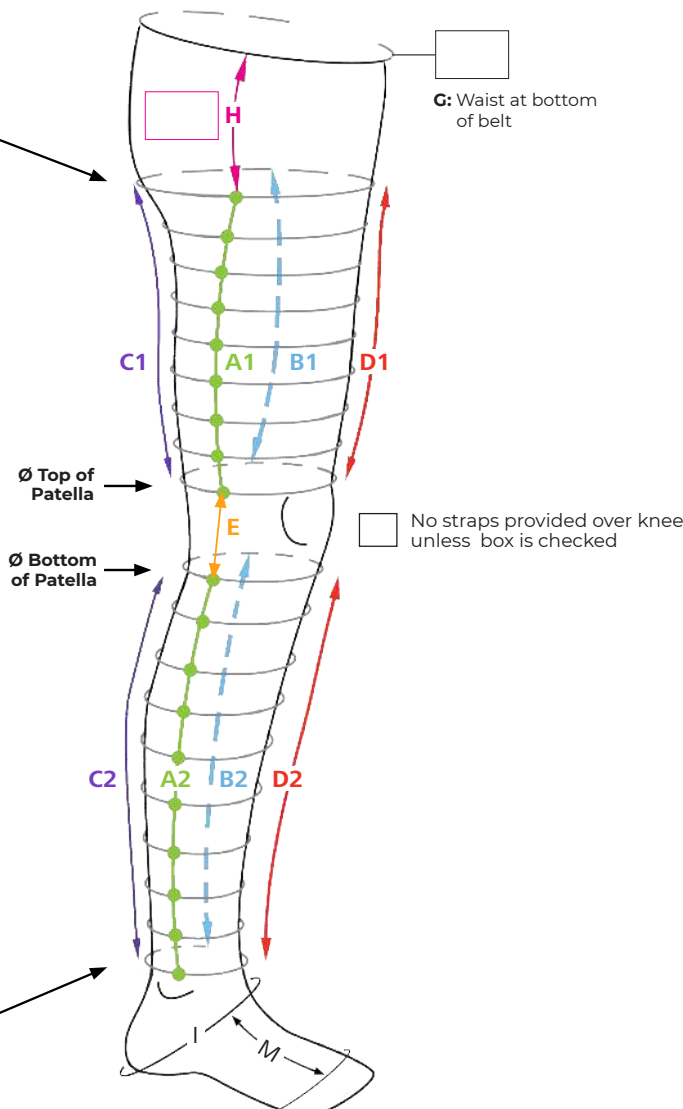

### Length Measurements

#### Lateral

A1 \_\_\_\_\_

A2 \_\_\_\_\_

#### Medial

B1 \_\_\_\_\_

B2 \_\_\_\_\_

#### Posterior

C1 \_\_\_\_\_

C2 \_\_\_\_\_

#### Anterior

D1 \_\_\_\_\_

D2 \_\_\_\_\_

#### Knee

E \_\_\_\_\_

### Foot Measurements

I \_\_\_\_\_

J \_\_\_\_\_

K \_\_\_\_\_

M \_\_\_\_\_

sigvaris

Legassist®  
FULL LEG

## CUSTOM Measuring Instructions

### Important

Exact measurements are critical for this garment to ensure a proper fit. If you have questions about measuring for this garment or would like assistance with measuring and fitting this garment please contact

### Supplies needed

- Cell phone with camera. Photos of the arm with measurement markings must be emailed to:
- SIGVARIS GROUP Measuring tape and body pen (or eyeliner pencil).
- Measuring instructions and forms.

## INSTRUCTIONS

All measurements should be recorded in centimeters. Apply slight tension to hold the tape measure in place.

### STEP 1: Locate Landmarks

- Encircle leg with top edge of tape measure located at the bottom of patella. Mark leg on this edge of tape at the anterior, lateral, posterior and medial aspects, recording the circumference on the line labeled bottom of patella (Ø point).
- Encircle leg with the bottom edge of the tape measure aligned with the ankle bend. Mark leg on this edge of the tape at the anterior, lateral, posterior and medial aspects, recording the circumference on the line labeled ankle bend.
- Encircle leg with bottom edge of the tape measure aligned with the top of patella. Mark leg on this edge of the tape at the anterior, lateral, posterior and medial aspects, recording the circumference on the line labeled top of patella.
- You may choose to end the garment circumferentially at the gluteal crease, or extend the garment proximally on the lateral aspect of the thigh to the greater trochanter. Mark your preferred position for the top of the garment along the medial thigh, at least 4cm distal to the groin. Next, mark your preferred position for the top of the garment along the lateral thigh, anywhere from the level of the gluteal fold to the lateral greater trochanter. Encircle the leg with the proximal edge of the tape measure positioned just below your markings, trace a line around the leg along the top edge of the tape with your body pen, and record the circumference on top of garment line.

### STEP 2: Locate Circumference Measurement Positions

- Starting with "0" of the tape measure at your mark on the lateral aspect of the bottom of patella (Ø point), mark the leg in 5cm increments down to your mark on the lateral aspect of the ankle bend.
- Repeat step 1, only this time from the top of the patella (Ø point) up to the lateral aspect of the top of garment.

### STEP 3: Measure Lengths

#### Lateral (A1) Lengths:

Measure upper leg from your mark on the lateral aspect of top of patella to your mark on lateral aspect of top of garment. Record length on line A1.

#### Medial (B1) Lengths

Measure upper leg from your mark on medial aspect of top of patella to your mark on medial aspect of top of garment. Record length on line B1.

#### Posterior (C1) Lengths

Measure upper leg from your mark on posterior aspect of top of patella to your mark on posterior aspect of top of garment. Record length on line C1.

#### Anterior (D1) Lengths

Measure upper leg from your mark on anterior aspect of top of patella to your mark on anterior aspect of top of garment. Record length on line D2.

# sigvaris

# Legassist®

**FULL LEG**

## CUSTOM Measuring Instructions

### Lateral (A2) Lengths:

Measure lower leg from your mark on the lateral aspect of bottom of patella to your mark on lateral aspect of ankle bend. Record length on Line A2.

### Medial (B2) Lengths

Measure lower leg from your mark on medial aspect of bottom of patella to your mark on medial aspect of ankle bend. Record length on Line B2.

### Posterior (C2) Lengths

Measure lower leg from your mark on posterior aspect of bottom of patella to your mark on posterior aspect of ankle bend. Record length on line C2.

### Anterior (D2) Lengths

Measure lower leg from your mark on anterior aspect of bottom of patella to your mark on anterior aspect of ankle bend. Record length on line D2.

### Knee Space

Measure leg from your mark on the lateral aspect of top of patella to your mark on lateral aspect of bottom of patella. Record length on line E.

## STEP 4: Measure And Record Circumferences

**Upper:** Encircle leg with tape measure at each mark located on lateral aspect between the top of patella and top of garment, and record each circumference on the corresponding line.

**Knee:** Encircle leg with tape measure at mid patella and record each circumference on the corresponding line.

**Lower:** Encircle leg with tape measure at each mark located on the lateral aspect between bottom of patella and ankle bend, and record each circumference on the corresponding line.

## STEP 5: Measure Foot

1. Measure foot from heel to 1st metatarsal head (or desired boot length) and record length on line K.
2. Measure top of foot from the 3rd metatarsal head to ankle bend and record length on line M.
3. Encircle the instep of foot and apex of heel with a tape measure and record the circumference on line I.
4. Encircle the foot across the metatarsal heads and record the circumference on line J.

## Hip Attachment (Optional)

1. Measure from lateral aspect of gluteal fold to lateral aspect of hip at the desired location for the bottom of the belt, and record length on line H.
2. Measure circumference of torso at the desired location for the bottom of the belt, and record circumference on line G.

NOTES: Legassist Full Leg is available in:

7099-TH Flat Foam, 7199-TH WaveFoam, 7299-TH Super Flat Foam, 7399-TH Super WaveFoam, TH with Hip Attachment is available in 7099-TA Flat Foam, 7199-TA Wave Foam, 7299-TA Super Flat Foam, 7399-TA Super WaveFoam (Includes MedaBoot Custom 7099-MB)

sigvaris

Legassist®  
LOBULE COMPRESSION SYSTEM (LCS)

|                      |               |               |            |         |
|----------------------|---------------|---------------|------------|---------|
| Patient Name:        | Contact Name: | Height:       | Weight:    | Date:   |
| Ship Name & Address: |               | Account Name: | Account #: | P.O #.: |


### Product Information

Product includes one Legassist Lobule Compression System (LCS).

|                                                                       |                                                                       |                                                                    |                                         |
|-----------------------------------------------------------------------|-----------------------------------------------------------------------|--------------------------------------------------------------------|-----------------------------------------|
| <input type="checkbox"/> Right Leg                                    | <input type="checkbox"/> Left Leg                                     | Foam                                                               | Optional                                |
| Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | <input type="checkbox"/> Hip Attachment |
| <input type="checkbox"/> Right Leg                                    | <input type="checkbox"/> Left Leg                                     | Foam                                                               | Optional                                |
| Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | Size: <input type="checkbox"/> Regular <input type="checkbox"/> Super | <input type="checkbox"/> Flat<br><input type="checkbox"/> Wavefoam | <input type="checkbox"/> Hip Attachment |

### Note:

If the greatest circumference measurement is >90cm, order a Super.

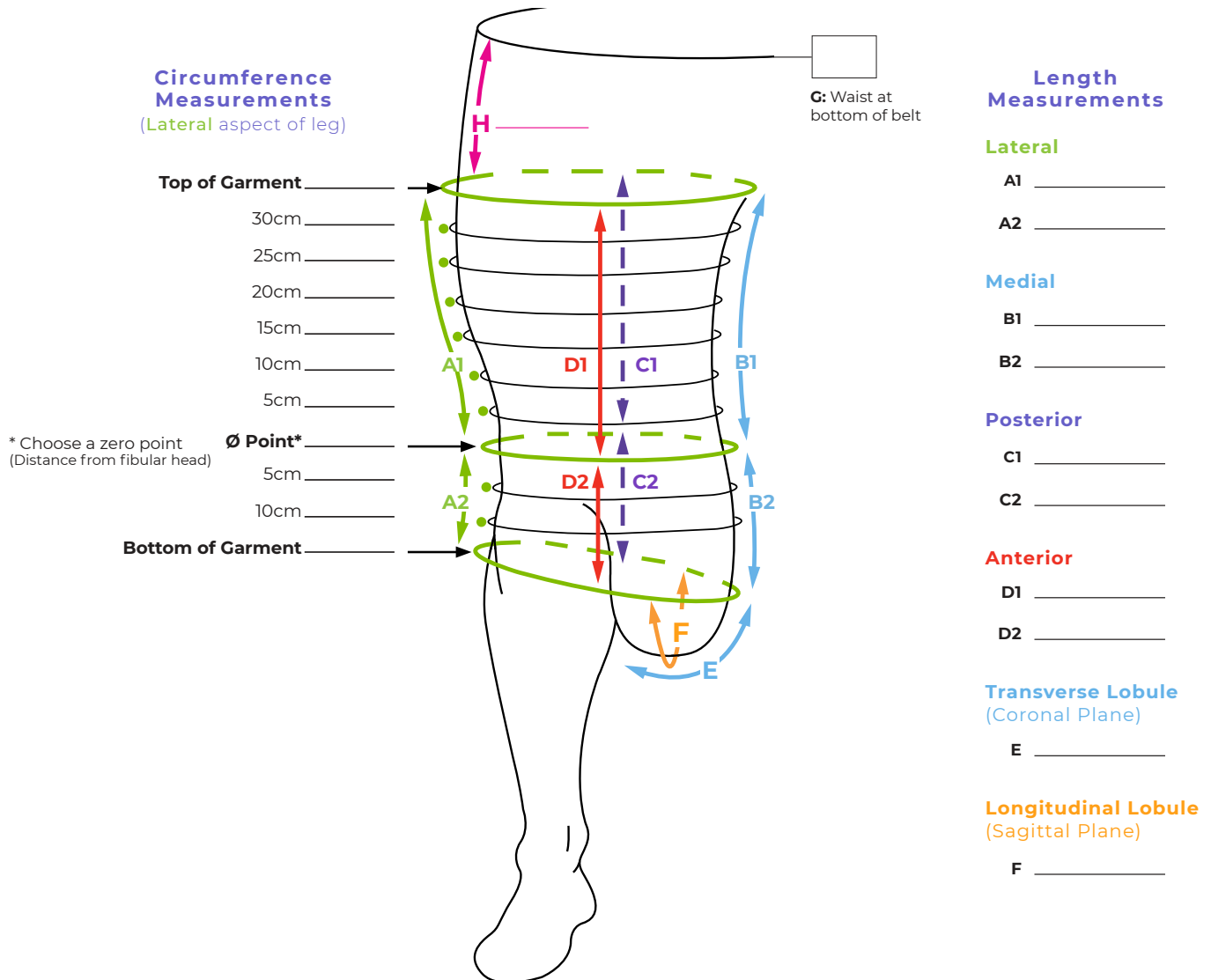

sigvaris

Legassist®  
LOBULE COMPRESSION SYSTEM (LCS)

## CUSTOM Measuring Instructions

### Important

Exact measurements are critical for this garment to ensure a proper fit. If you have questions about measuring for this garment or would like assistance with measuring and fitting this garment please contact

### Supplies needed

- Cell phone with camera. Photos of the arm with measurement markings must be emailed to:
- SIGVARIS GROUP Measuring tape and body pen (or eyeliner pencil).
- Measuring instructions and forms.

## INSTRUCTIONS

All measurements should be recorded in centimeters. Apply slight tension to hold the tape measure in place.

### STEP 1: Choose The Zero Point (Ø point)

The zero point is unique to the Legassist LCS garment. It is the reference point from which all other measurements are taken, in order to determine the lengths needed for the custom garment to fit the patient's individual leg shape. Choose a location proximal to the knee joint where the leg shape is the most cylindrical and the tape measure can be positioned circumferentially around the leg and exactly parallel to the floor. Trace a line around the leg at this Zero Point using the SIGVARIS body pen and record the circumference on Ø point line. If possible, document the location of the zero point by measuring the distance from the head of the fibula to the zero point.

### STEP 2: Choose Where The Garment Will End Distally - "bottom of garment"

The length of the Legassist LCS will depend on how low the medial knee lobule hangs down. Start by marking the lowest aspect of the curve of the lobule where a tape measure could be applied without sliding off (lobule apex). Next, choose and mark a point on the lateral aspect of the leg where the garment will overlap the lower leg piece by at least 3 centimeters and create no more than a 30degree downward angle or no more than 10degree upward angle to the lobule apex. Encircle the leg from this point to the lobule apex, with the tape measure positioned just above your marking points. Trace this circumference with your body pen along the bottom edge of the tape and record the circumference measurement on bottom of garment line.

### Problem Solving

If the bottom of garment is proximal to the knee, due to a lobule positioned very high on the thigh, you will need to measure for a customized Legassist Full leg system that overlaps with the Legassist LCS.

If the bottom of garment extends down to the mid-lower leg or ankle, due to a lobule positioned very low on the leg, you will need to measure for a customized Legassist Calf & Foot that is shortened to best overlap with the Legassist LCS.

### STEP 3: Choose Where The Garment Will End Proximally - "top of garment"

You may choose to end the garment circumferentially at the gluteal crease, or extend the garment proximally on the lateral aspect of the thigh to the greater trochanter. Mark your preferred position for the top of the garment along the medial thigh, at least 4cm distal to the groin. Next, mark your preferred position for the top of the garment along the lateral thigh, anywhere from the level of the gluteal fold to the lateral greater trochanter. Encircle the leg with the proximal edge of the tape measure positioned just below your markings, trace a line around the leg along the top edge of the tape with your body pen, and record the circumference on top of garment line.

### STEP 4: Mark the Midline of Garment and Record Lengths

Step back to visualize the leg and your marking points to determine where the midline of the garment would be, medial to lateral, encompassing the lobules. Mark the midline of the garment anteriorly (D) and posteriorly (C) at the top of garment, zero point, and bottom of garment lines. Measure each of the lengths from the zero point, following the contour of the leg but not extending the tape measure into folds. Record lengths as follows:

sigvaris

Legassist®  
LOBULE COMPRESSION SYSTEM (LCS)

## CUSTOM Measuring Instructions

### Lateral (A) Lengths:

A1: Measure & record lateral length from Ø point to top of garment

A2: Measure & record lateral length from Ø point to bottom of garment

### Medial (B) Lengths

B1: Measure & record medial length from Ø point to top of garment

B2: Measure & record medial length from Ø point to bottom of garment

### Posterior (C) Lengths

C1: Measure & record posterior length from Ø point to top of garment

C2: Measure & record posterior length from Ø point to bottom of garment

### Anterior (D) Lengths

D1: Measure & record anterior length from Ø point to top of garment

D2: Measure & record anterior length from Ø point to bottom of garment

## STEP 5: Measure And Record Circumferences

Starting from the zero point, on the lateral aspect of the leg only, mark every 5cm proximally from the zero point and mark every 5 cm distally from the zero point. Record the circumferences at each point, making sure that the measuring tape is parallel to the floor (Note that some of the spaces to record circumferences may not be used, or you may need to add additional spaces, depending on the length of the limb).

## STEP 6: Measure Foot

Measure the size of the lobule for the part of the LCS system that will lift and compress the lobule, called the "Lobule Sling".

Starting at the medial aspect of the lobule, measure the distance from the bottom of garment line to the medial aspect of the leg. Follow the curve of the lobule only to where it touches the leg. Do not extend the tape measure up into the crevices between the lobule and the leg. Record this length as E. At the midline of the lobule, measure from the anterior line of the bottom of garment, underneath the lobule and up to the posterior line of the bottom of garment. Record this length as F.

## STEP 7: Hip Attachment (Optional)

1. Measure from lateral aspect of gluteal fold to lateral aspect of hip at the desired location for the bottom of the belt, and record length on line H.
2. Measure circumference of torso at the desired location for the bottom of the belt, and record circumference on line G.

## STEP 8: Pictures

Now that you have completed your measurements, take pictures of the leg (front, back, side) that show all of the measurement markings, as well as the overall shape of the leg. This will assist our expert design team to further customize the garment according to your patient's individual needs. E-mail the pictures with your completed forms to:

the garment.

## STEP 9: Select Additional Options

You are ready to complete your forms and place the order.

Choose right or left leg

Choose Size: If the greatest leg circumference is >90cm, order a super.

Choose Flat or Wavefoam (recommended)

Choose Hip Attachment, if desired (recommended)

## Fitting Tips

The ropey, thicker seam should be positioned laterally, and the tag is positioned at the top of the garment straps pull lateral to medial.